ProFem®

**Instructions for use (E)**  
**Gebruiksaanweisung (DE)**  
**Instructions d'utilisation (F)**  
**Instructies voor gebruik (NL)**  
**Brugsanvisning (DK)**  
**Käyttöohjeet (FI)**

**Instructions for use (E)**  
**for Silicone vaginal pessaries, type:**  
Ring pessary, with and without support  
Oval pessary, with and without support  
Donut Pessary  
Gellhorn Pessary, Long Stem  
Gellhorn Pessary, Short Stem  
Shaatz Pessary  
Cup Pessary, with and without support

**Indications for use**  
Flexible silicone pessaries of the above types are used to alleviate symptoms of uterine prolapse (prolapse symptoms), cystocele, rectocele or stress incontinence.

**Contraindications**  
The use of pessaries is contraindicated in patients with acute infections of the genital tract, pelvic inflammatory disease or endometriosis.

**Caution**  
This medical device should only be used on the instructions or prescription of a healthcare professional. Avoid any use of force when inserting or removing the pessary into the vagina. Improper use of pessaries can cause abrasion of the vaginal walls and ulceration. Pessaries are intended for use on one patient and are not supplied sterile.

The first time the pessary is used, the patient and doctor should insert it together. Clarify pessary use during sexual activity.

**Application**  
Before fitting and insertion, the doctor will determine the correct pessary through a general pelvic examination. She/he should determine if the patient requires oestrogen therapy before prescribing a pessary.

Generally, it may be more comfortable for the patient to use a vaginal lubricant before inserting the pessary.

Pessary fitting kits are a valuable aid in choosing the right size and type of pessary.

When fitting a pessary, the doctor will ensure that the patient is instructed in cleaning, insertion and removal. The patient should have performed these tasks before the first follow-up visit. It should also be noted that any discomfort should be reported to the doctor immediately.

The doctor will ensure within 24 to 48 hours that the patient is not allergic to pessaries. This includes examining the vagina and asking if the patient has experienced any discomfort, irritation, pressure, tenderness or unusual vaginal discharge. It is determined whether there has been any improvement in the patient's personal symptoms.

**Insertion**

- Familiarise yourself with the use of the pessary before using it for the first time.
- Squeeze or fold the pessary (2) and gently insert it into the vaginal opening (3).
- After the pessary has passed through the vaginal opening, release it so that it expands to its original shape.
- Gently push the pessary towards the cervix (4).
- When it is in the correct position, you should be able to insert a finger between the pessary and the vaginal wall. This distance ensures appropriate comfort and prevents pressure damage.
- After insertion, the pessary should remain permanently in position when standing, sitting, kneeling or bending. It should not be uncomfortable for you to wear during any of these routine activities.
- Should an allergic reaction, discomfort, irritation, pressure or unusual discharge occur after insertion of the pessary, the patient should consult her doctor immediately.

**Remove**

- Loosen the pessary.
- To pull out the pessary, gently push it forward towards the vaginal entrance. When the pessary is accessible, squeeze it and gently pull it out (5).

**Cleaning**

For cleaning, the pessary should be removed regularly. To do this, clean the pessary only with soapy water that is not too hot and then rinse carefully with warm water.

**Safety Control Check**

Check the pessary regularly for cracks and damage! Replace the defective pessary with a new one immediately, as it can no longer be cleaned properly.

**Recommendation**

Patient and doctor should make appointments for follow-up examinations early.

For more information and our local distributors visit: **www.profem.eu**

**LOT**

Charge Batch code

**!**

Achtung! Attention

**LATEX**

Latexfrei Latex free

**!**

Verfalldatum Expiry date

**CH REP**

CH I REP

**Expirion GmbH (CH)**

Grabenhofstrasse 1

CH-6010 Kriens 2

Tel. +41 360 27 64

Fax +41 360 27 18

info@expirion.ch

www.profem.eu

**SAYCO PTY Ltd. (AU)**

Unit 5/260 Wickham Road

Moorabbin Victoria 3189

Tel. +61 3 9553 2700

Fax +61 3 9553 4858

Toll free: 1800 333 456

customerservice@sayco.net.au

www.profem.com.au

**Distribution:**

**Sayco Europe GmbH (DE)**

Posener Straße 48

74321 Bietigheim-Bissingen

Tel. +49 7142 913 22 33

Fax +49 7142 779 24 28

info@sayco-europe.com

www.profem.eu

**GUANGZHOU FAME MEDICAL CO., LTD.**


Second Floor A No. 8, Lianhua Port Industrial Zone

Lotus Mountain Bonded Area,


Shilou Town Panyu District,

Guangzhou China


www.famebiology.com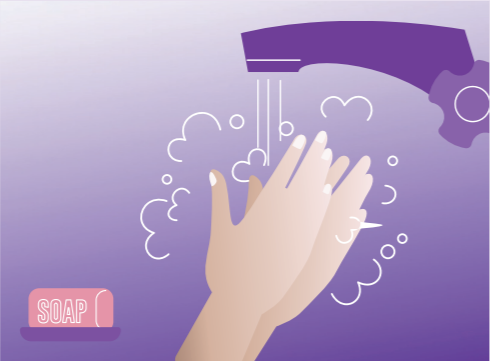


1. Washing hands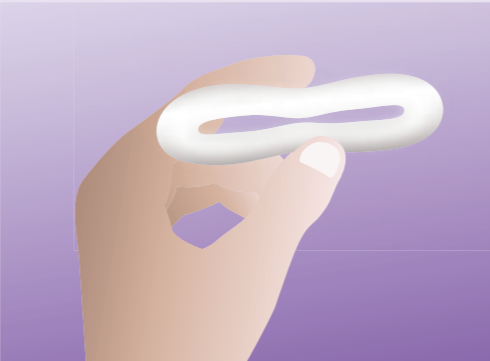


2. Fold the pessary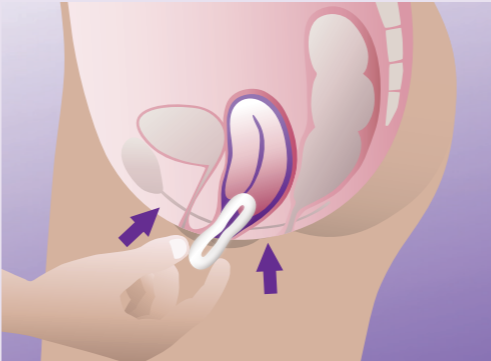

3. Insert pessary and push up with finger alternately front and back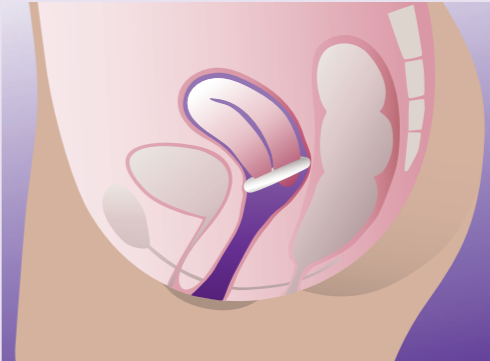

4. Correct positioning of the pessary

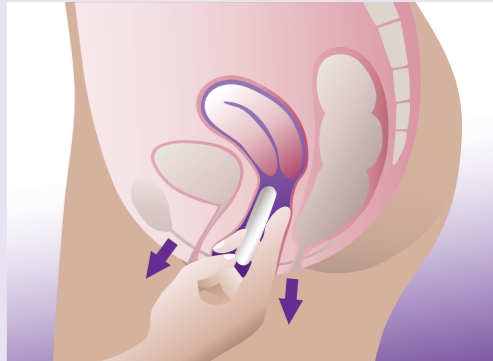5. Grasp pessary with two fingers and pull out towards vaginal opening

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